



Horizontal Autoclave Sterilizer Catalog

Picture	Description	Model No.	Volume	Product Dim. (mm)	Gross Weight (kg)
	<p>Features:</p> <ol style="list-style-type: none"> 1. The steam can't go through to chamber when the door haven't close, and the pilot off inform. You can processing dry operate. 2. When the pressure inner over 0.027MPa. The door be locked and can't be open. 3. If you not choose the sterile time and star the machine, buzzer sound. 4. If you mistake open the door in the course of start, the jackervalue auto off and open the chamber value. 5. Lack water the pilot on, heating and sterile pilot off, the buzzer sound and auto release steam. 6. Overload and leakage protection, when leakage or over current occur, it auto off the power. 7. Temperature controlled not work and the inner pressure over 0.165MPa, the safety value auto open. <p>Technical Data:</p> <p>Working pressure: 0.22Mpa Working temperature:134°C Adjustment of temperature:40°C-134°C Timer: 0-60min Drying Time Range: 0-60min Power: 9-18KW / AC220V 50HZ</p>	HA-BC150	150L Ø400*1000mm	1400*600*1300	320
		HA-BC200	200L Ø515*1000mm	1400*670*1650	350
		HA-BC280	280L Ø600*1000mm	1400*770*1780	465
		HA-BC400	400L Ø700*1100mm	1430*880*1830	530
		HA-BC500	500L Ø700*1300mm	1800*900*1820	580


Please find your preferred models and send email to us for request Latest Best Price

Horizontal Autoclave Sterilizer Catalog

Picture	Description	Model No.	Volume	Product Dim. (mm)	Gross Weight (kg)
	<p>Features:</p> <ol style="list-style-type: none"> 1. The steam can't go through to chamber when the door haven't close, and the pilot off inform. You can processing dry operat. 2. When the pressure inner over 0, 027MPa. The door be locked and can't be open. 3. If you not choose the sterile time and star the machine, buzzer sound. 4. If you mistake open the door in the course of start, the jackervalue auto off and open the chamber value. 5. Lack water the pilot on, heating and sterile pilot off, the buzzer sound and auto release steam. 6. Overload and leakage protection, when leakage or over current occur, it auto off the power. 7. Temperature controlled not work and the inner pressure over 0.165MPa, the safety value auto open. <p>Technical Data:</p> <p>Working pressure: 0.22Mpa Working temperature: 134°C Adjustment of temperature: 40~134°C Timer: 0~60min Drying Time Range: 0-60min Power: 9-18KW / AC220V 50HZ</p>	HA-BD150	150L Ø440*1000mm	1400*600*1300	320
		HA-BD200	200L Ø515*1000mm	1400*670*1650	350
		HA-BD280	280L Ø600*1000mm	1400*770*1780	465

Please find your preferred models and send email to us for request Latest Best Price

Horizontal Autoclave Sterilizer Catalog

Picture	Description	Model No.	Volume	Product Dim. (mm)	Gross Weight (kg)
	<p>Features:</p> <ol style="list-style-type: none"> 1. Sterilizing course automatic sterilization controlled, easy to operate. 2. With drying function, suitable dressing drying. 3. With over-temperature, over-pressure auto-protect device. 4. The door opening mechanism cannot be operated until the pressure in the chamber is reduced to 0.027 Mpa. It can't be start on if the door doesn't close well. 5. The safety valve will be open automatically when the inner pressure over 0.24 MPa and the steam be exhausting to the water link. 6. The power will be cut-off automatically, cut-off the water and alarm when the machine lacking water. 7. Fully stainless steel structure. 8. With printer. <p>Technical Data:</p> <p>Working pressure: 0.22Mpa Working temperature: 134℃ Adjustment of temperature: 40~134℃ Timer: 0~99min Drying Time Range: 0-99min Power: 9-12KW / AC380V 50HZ</p>	HA-BA150	150L Ø440*1000mm	1400*650*1600	430
		HA-BA200	200L Ø515*1000mm	1400*750*1700	436
		HA-BA280	280L Ø600*1000mm	1520*910*1900	570

Please find your preferred models and send email to us for request Latest Best Price



Kayla Africa Suppliers & Distributors Cc


Address: Unit C9, Wadeville Business Park, Cnr Steenbras & Shad Roads, Wadeville, Germiston, Johannesburg

Email: k.a.s.d@telkomasa.net

Website ID: www.kasd.co.za


Tel: +2711 027 7221, Mobile / WhatsApp : +27 83 488 5196

Horizontal Autoclave Sterilizer Catalog

Picture	Description	Model No.	Volume	Product Dim. (mm)	Gross Weight (kg)
	<p>Features:</p> <ol style="list-style-type: none"> 1. Sterilizing course automatic sterilization controlled, easy to operate. 2. With drying function, suitable dressing drying. 3. With over-temperature, over-pressure auto-protect device. 4. The door opening mechanism cannot be operated until the pressure in the chamber is reduced to 0.027 Mpa. It can't be start on if the door doesn't close well. 5. The safety valve will be open automatically when the inner pressure over 0.24 MPa and the steam be exhausting to the water link. 6. The power will be cut-off automatically, cut-off the water and alarm when the machine lacking water. 7. Fully stainless steel structure. 8. With printer. <p>Technical Data:</p> <p>Working pressure: 0.22Mpa Working temperature: 134°C Adjustment of temperature: 40~134°C Timer: 0~99min Drying Time Range: 0-99min Power: 9-12KW / AC380V 50HZ</p>	HA-BB150	150L Ø440*1000mm	1400*650*1600	430
		HA-BB200	200L Ø515*1000mm	1400*750*1700	436
		HA-BB280	280L Ø600*1000mm	1520*910*1900	570

Please find your preferred models and send email to us for request Latest Best Price

Horizontal Autoclave Sterilizer Catalog

Picture	Description	Model No.	Volume	Product Dim. (mm)	Gross Weight (kg)
	<p>Features: HA-BVD Double Door Pulse Vacuum Autoclave 1. Adopts the programmable controller (Siemens PLC) to control the entire cycle, and also with manual operation and automatic operation modes for choosing 2. Piping system: Adopt with the reliable Germany Burket or GEMU pneumatic angle type valve, water ring vacuum pump 3. 0.22µm high efficiency sterile medical grade filters. 4. The chamber and the steam piping system is made of high quality stainless steel. 5. All alarm and technical figures can be recorded and printed 6. If pressure of inner chamber is higher than standard, door will not open 7. Operating authority is divided into three levels: Manager, technologist, operator. Both Inside and Outside is Full Stainless steel material. Note: From 250L to 600L, the price include the internal steam generator. From 250L to 2500L include the price of external air compressor. From 800L to 2500L, need external steam generator, the price do not include it.</p> <p>Technical Data: Working pressure: 0.22Mpa Working temperature: 134°C Adjustment of temperature: 105~134°C Timer: 0~99min Power: 12-16KW / AC220V 50HZ</p>	HA-BVD250	250L 800*600*600	1100*1350*1900	900
		HA-BVS250	250L 800*600*600	1100*1350*1900	900
		HA-BVD360	360L 1000*600*600	1300*1350*1900	1000
		HA-BVS360	360L 1000*600*600	1300*1350*1900	1000
		HA-BVD600	600L 1200*610*910	1500*1360*1950	1400
		HA-BVS600	600L 1200*610*910	1500*1360*1950	1400
		HA-BVD800	800L 1500*610*910	1800*1360*1950	1600
		HA-BVS800	800L 1500*610*910	1800*1360*1950	1600
		HA-BVD1200	1200L 1450*680*1180	1750*1440*1950	1800
		HA-BVS1200	1200L 1450*680*1180	1750*1440*1950	1800
		HA-BVD1500	1500L 1850*680*1180	2150*1440*1950	2000
		HA-BVS1500	1500L 1850*680*1180	2150*1440*1950	2000
		HA-BVD2000	2000L 1600*900*1400	1950*1750*2200	2500
		HA-BVS2000	2000L 1600*900*1400	1950*1750*2200	2500
		HA-BVD2500	2500L 1950*900*1400	2300*1750*2200	3000
HA-BVS2500	2500L 1950*900*1400	2300*1750*2200	3000		

Please find your preferred models and send email to us for request Latest Best Price



Kayla Africa Suppliers & Distributors Cc

Address: Unit C9, Wadeville Business Park, Cnr Steenbras & Shad Roads, Wadeville, Germiston, Johannesburg

Email: k.a.s.d@telkomasa.net

Website ID: www.kasd.co.za

Tel: +2711 027 7221, Mobile / WhatsApp : +27 83 488 5196